



Ready, Steady, EnGO!
The wireless handpiece
with file database.





Brasseler®, Komet®, Art2®, CeraBur®, CeraCut®, CeraDrill®, CeraPost®, DC1®, DIAO®, FastFile®, F360®, F6 SkyTaper®, H4MC®, KometBioSeal®, OccluShaper®, OptiPost®, PolyBur®, PrepMarker®, Procodile®, Procodile Q® and SHAX® are registered trademarks of Gebr. Brasseler GmbH & Co. KG.

ReFlex® and EndoPilot® are registered trademarks of the company Schlumbohm GmbH & Co. KG.

Some of the products and designations mentioned on our web sites are protected by patent law, trademark law and copyright. The absence of an express statement or of the symbols ® should not be interpreted as absence of protection.

This publication is copyrighted. All rights, also with regard to translation, reprint and reproduction (also in the form of extracts) are reserved. No part of this publication may be reproduced or reprocessed using electronic systems in any form or by any means (photocopying, microfilm or other methods) without the written permission of the editor.

Colours and products are subject to alterations.
Printing errors excepted.

As at January 2023

Flexible. Safe. Powerful.

EnGO – for comfort in endodontics.

With Komet's EnGO handpiece, you have a true multi-talent for endodontics in your hands. Whether reciprocating or rotating: EnGO is suited for all common types of movements and offers individual setting options for speed, torque, and angle. The specifications for your most frequently used files can be saved in the unit. Thanks to the pre-installed parameters of all Komet systems, you are all set for your next treatment.

The wireless, ergonomic unit fits perfectly into the hand of both right-handed and left-handed users and makes work more comfortable, even during longer treatments. Thanks to its special contra-angle that can be rotated by 360°, EnGO provides optimal access to the treatment area. In addition, EnGO is particularly safe to use: the small head ensures good visibility, and the integrated apex locator allows you to prepare right up to the apex.



ENGO13

With EnGO through endo.

The wireless handpiece with programmable file memory.

Individual settings.

The integrated memory holds up to ten different movement programs. Speed, torque, and angle can be individually adjusted to suit each file. The parameters of all Komet systems are already pre-installed – so you are ready for your next treatment.

Sophisticated Ergonomics.

Whether right- or left-handed: the integrated LCD display can be adjusted to your personal needs. Furthermore, EnGO fits perfectly into your hand for minimum fatigue during work. The balanced weight distribution of the unit further reduces stress to your hand and wrist.



Optimum control during the entire treatment.

Integrated Apex Locator.

Thanks to the integrated apex locator, your treatment becomes even safer.

The automatic length measurement shows you in real time how far you have advanced towards the apex thus minimizing the risk of overpreparation.



Optimum View.

Throughout the treatment, the narrow contra-angle head provides a particularly large field of view of the treatment area. The treatment is further facilitated by the 360° rotation of the contra-angle head which allows optimum access to the treatment site.



That's not all yet.

These additional EnGO features round off your treatment:

- **Impressive Performance**

With up to 2,500 revolutions per minute, EnGO opens up further areas of application – while keeping its high precision.

- **Rotatable all around**

The 360° rotation of the contra-angle provides optimum access to the treatment area.

- **Automatic Movement**

For more control: EnGO starts automatically as soon as you insert the file into the root canal and stops once you remove the file from the canal.

- **Adjustable Reciprocating Movement**

The reciprocating angle can be individually adjusted in intervals of 10° to suit any customary file.

- **Easy Handling**

Through its ergonomic shape and smooth operation, EnGO ensures maximum control during treatment.

- **Wireless Charging**

Whether during work or while charging up, EnGO offers maximum flexibility thanks to its wireless design.

- **Long-lasting Battery**

Thanks to the powerful battery, you can always rely on EnGo's durable strength to last through the entire treatment – even if it takes a little longer than expected.



Get started straight away.

Thanks to the pre-installed settings for your Komet instruments.

The parameters of all Komet systems are already pre-installed.

You can also save your own settings.

FQ files



FQ Opener
Operating mode CW
Torque 2,4 Ncm ⌚_{opt.} 300 rpm



FQ Glider
Operating mode CW
Torque 0,8 Ncm ⌚_{opt.} 300 rpm



FQ04 020 - 035
Operating mode CW
Torque 1,8 Ncm ⌚_{opt.} 300 rpm



FQ04 045 - 055
Operating mode CW
Torque 2,4 Ncm ⌚_{opt.} 300 rpm



FQ06 020 - 030
Operating mode CW
Torque 2,4 Ncm ⌚_{opt.} 300 rpm

Procodile Q files



Rez. Opener
Operating mode RECIP
Torque Auto



Procodile Q 020 - 050
Operating mode RECIP
Torque Auto

Other rotary files



Opener
Operating mode CW
Torque 2,5 Ncm ⌚_{opt.} 300 rpm



PathGlider
Operating mode CW
Torque 0,5 Ncm ⌚_{opt.} 300 rpm



F360
Operating mode CW
Torque 1,8 Ncm ⌚_{opt.} 300 rpm



F6 SkyTaper 020 - 030
Operating mode CW
Torque 2,2 Ncm ⌚_{opt.} 300 rpm



F6 SkyTaper 035 - 040
Operating mode CW
Torque 2,8 Ncm ⌚_{opt.} 300 rpm

Retreatment file



Endo ReStart
Operating mode CW
Torque 1,8 Ncm ⌚_{opt.} 300 rpm

Komet Dental

Gebr. Brasseler GmbH & Co. KG
Trophagener Weg 25 · 32657 Lemgo
Postfach 160 · 32631 Lemgo · Germany

Verkauf Deutschland:

Telefon +49 (0) 5261 701-700
Telefax +49 (0) 5261 701-289
info@kometdental.de
www.kometdental.de

Komet Austria Handelsagentur GmbH
Hellbrunner Straße 15
5020 Salzburg · Austria

Telefon +43 (0) 662 829-434
Telefax +43 (0) 662 829-435
info@kometdental.at
www.kometdental.at

Export:

Telefon +49 (0) 5261 701-0
Telefax +49 (0) 5261 701-329
export@kometdental.de
www.kometdental.com
